



# SARS-CoV-2 IgM/IgG Antibody Test Kit


## (colloidal gold method)

The kit is immunochromatographic and uses capture method to detect SARS-CoV-2 IgM and IgG antibodies. During detection, the specimens are loaded to the sample wells of the test card. IgM and IgG can be detected simultaneously; this kit can cover the middle and late stages of virus infection; According to the test results, the incidence of patients can be judged; the cost of double antibody detection is relatively low compared with other detection methods.


**1** Using Bioer SARS-CoV-2 IgM/IgG Antibody Test Kit (colloidal gold method)




**2** 10ul to the sample well

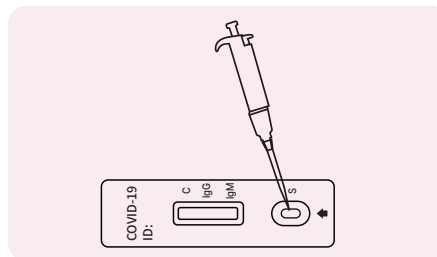


**3** 2 drops of sample diluent

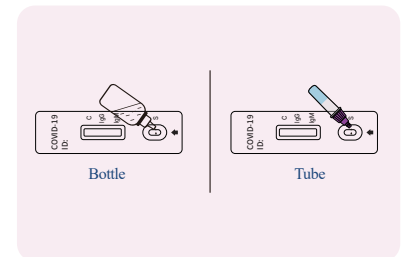

**4** Wait for 15min

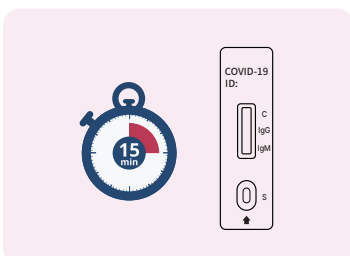

**5** The detection results of the kit are interpreted as above.

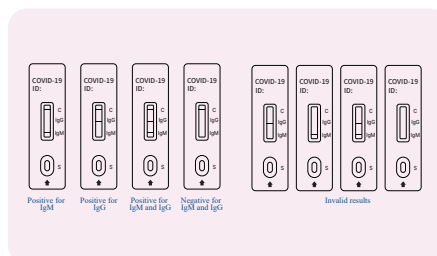



## Specifications

[www.bioer.com.cn](http://www.bioer.com.cn)

Components	Test card、 Sample diluent
Pack Size	25Tests/Kit; 50 Tests/Kit
Storage Conditions	2°C~30°C ;Valid for 18 months
Test samples	Human serum or plasma
Reaction conditions	Room temperature 15min
Special instrument	No need

## Features

[www.bioer.com.cn](http://www.bioer.com.cn)

Early diagnosis	Diagnosis can be made within 1 to 7 days after infection
Easy to use	It takes only a few simple steps to complete the detection
High safety	Home inspection can be realized to reduce the risk of infection
High specificity	No cross reaction with other viruses
Rapid diagnosis	Only 15min for results

## Ordering Information

[www.bioer.com.cn](http://www.bioer.com.cn)

Product Name	Cat#	Package	Sample Type
SARS-CoV-2 IgM/IgG Antibody Test Kit (colloidal gold method)	BSK02S1	25 Tests/Kit	Human serum or plasma
	BSK02S1S	25 Tests/Kit	
	BSK02M1	50 Tests/Kit	
	BSK02M1S	50 Tests/Kit	

**Note:** BSK02S1 contains bottled sample diluent; BSK02S1S contains tubes of sample diluent. Same as BSK02M1&BSK02M1S



**HANGZHOU BIOER  
TECHNOLOGY CO., LTD.**

**Address:** 1192 Bin 'an Road, Hi-tech (Binjiang) District, Hangzhou, 31005, P.R.China  
**Direct Phone:** +86-571-87774575 (Overseas Department) **Fax:** +86-571-87774553  
**Website:** <http://www.bioer.com.cn> **Email:** [overseas@bioer.com.cn](mailto:overseas@bioer.com.cn) **E-Date:** 2022.01  
 \* All rights reserved BIO K02-2201

